Via Ugo Foscolo 21/A  
57025 Piombino (LI)  
ITALIA  
Tel. +39 370 3432913  
info@talitha-info.com  
www.talitha-info.com

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Dott. Matteo Tonietti

# Instructions for the use of the Oxalic Acid Vaporizer OXALIKA

EN

Read the instructions carefully before using the Oxalic Acid Vaporizer and always store it in a safe place.

## INDEX

- I. Important Safety Instructions
- II. Specifications and functions
- III. Treatment and methods of use
- IV. Storage
- V. Maintenance
- VI. Product packaging and disposal
- VII. Product liability limits
- VIII. Declaration of conformity

### I. Important Safety Instructions

To reduce the risk of fire, electric shock, injury, burns, and oxalic acid exposure damage, follow the basic precautions described below.

1. Read all instructions before using the appliance
2. Only use the unit for the proposes described in the manual.
3. Do not use the appliance on hives that are not populated by bees.
4. If the power cord is damaged, request a replacement from the producer, production technician assistant, or any other qualified technician to avoid all dangers.
5. This device is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they are subjected to supervision or instruction concerning use of the appliance by a person responsible for their safety and understand the dangers of improper use.
6. Do not let children play with the appliance
7. **To reduce the risk of fire and electric shock:**
  - a. Do not directly connect to an electrical outlet of 110-220 volts
  - b. The product must be powered by a 12 volt battery.
  - c. Do not use the product with a damaged battery
  - d. While using the vaporizer and the battery, they must be kept away from any flammable material. Pay attention to the electric arc that forms while the power cables are connected to the battery terminals.
  - e. Do not use the product in the case of breakage or damaged components of the product.
8. **To reduce the risk of burns:**
  - a. The cup of the burner reaches temperatures of about 400 °C (Oxalika Basic/Start), 200 °C (Oxalika Premium), which puts you at risk to be burned.
  - b. Make sure no one touches the hot appliance.

### 9. To reduce the risk of damage caused by oxalic acid exposure:

- a. Do not use in enclosed spaces
- b. Only use outdoors and in well-ventilated areas.
- c. Do not breathe in the vapors and dust.
- d. Wear a suit suitable to protect the entire body from the possible vapor or contact with oxalic acid powder when filling the cup and during the treatment.
- e. Pay particular attention to the manipulation of oxalic acid.
- f. avoid oxalic acid contact with skin, eyes and the respiratory track system

- g. Wear adherent goggles  
Follow the directions given in the respective user manuals.

- h. Wear gloves  
Follow the directions given in the respective user manuals.

- i. Wear a face mask to protect the respiratory system:
  - Type FFP3 in case of sufficient ventilation and using the vaporizer in the first mode (chapter. III)
  - With a filter Polyvalent ABE/P3 in case of insufficient ventilation or for prolonger exposure or using the vaporizer in the second mode (chapter. III)

Follow the directions given in the respective user manuals.

- l. Always keep in mind the direction of the wind so that the oxalic acid will not be blown towards any person including those who perform the treatment. Keep it as far away as possible from the hive, it is usually most advisable to use the entire length of the cable.
- m. The treatment of the colonies in an apiary should be done only from the outside. During and after the treatment ventilate the apiary well. Nothing should be inside.
- n. Do not extract the dispensing device from the hive before the time is complete and the oxalic acid is evaporated.
- o. The foam or fabric, used to close the opening and to seal the hive, must be handled with gloves, because the oxalic acid will be put inside the hive with it. After use, wash with water.
- p. After the treatment, especially after the winter treatment when the bees are not as clean as in the summer, inside the hive, in the drawer, there will be a diagnostic of oxalic acid crystals. Wear suitable protective gear and

use caution when handling.

- q. Keep the oxalic acid in its original packaging (with the label in accordance with the law) and keep it in a safe place.
- r. As a precaution, wash your hands thoroughly after use.
- s. **IF SWALLOWED:** Call a POISON CONTROL CENTER or doctor immediately.
- t. **IF CONTACT WITH EYES:** Rinse cautiously with water for several minutes. Remove contact lenses if necessary. Continue rinsing.
- u. **IN CASE OF CONTACT WITH SKIN:** wash thoroughly with plenty of water and soap.

**10. To reduce the risk of accidents and occupational diseases, in addition to the information given in the previous paragraphs:**

- a. Do not use the device without adequate lighting. In case, use artificial lighting systems.
- b. Clean the device only with water and prevent water from entering internally into the handle.
- c. Product use by professional operators who often intensely use the product must take account of any issues that may increase the exposure to the possible dangers.
- d. Do not leave the appliance unattended while it is on.
- e. Do not leave the device or the oxalic acid in environments where there are children or where children have access.
- f. Handle the device only by the handle
- g. For all other follow ups on the use of oxalic acid and security of the acid itself, you can find it provided in technical specifications.
- h. Given the nature of the device and its intended use, the dangers resulting specifically from the use of burners combine the dangers connected with the activity of apiculture. At such risks, the professional user must correct the same risks after the valuation opportunity of the current laws.

**11. To reduce the risk of damage:**

The cup of the burner reaches temperatures of about 400 °C (Oxalika Basic/Start), 200 °C (Oxalika Premium), which puts you at risk of damaging the parts of the hive in contact.

- a. Do not use on plastic or plastic-coated hive networks without applying a proper insulation (minimum 4 mm wooden board) under the cup
- b. To prevent burns on the wooden protector, apply a suitable insulator (minimum 4 mm wooden board) under the cup

**II. Specifications and Functions**

Oxalic acid is placed in the cup, which will be inserted into the opening. As the cup heats up, the oxalic acid evaporates. It is formed so that in the hive there is a mist that covers the bees and all surfaces with a thin layer of oxalic acid crystals. These small crystals are tolerated well by the bees, but they have

lethal effect on Varroa destructors.

The cup is connected to a solid grip where the cable extends, provided with terminals to connect to the accumulator (not provided) to be used as an energy source (battery for car, 12 volt, capacity > 30 Ah). The evaporator power is 130 Watt.

**III. Treatment and Methods of Use**

This equipment is intended for the use of oxalic acid vaporization suitable for the purpose of beekeeping.

The burner is designed and built to meet specific needs for the treatment of Varroa destructors in hives populated by bees.

The burner is used in treatments against colonies of mites, without hatching in November or December.

Given the high efficacy of the burner (> 96%), you can reduce the population of Varroa destructors so that the bees will be in an ideal condition for the next season.

**DOSE:** With Dadant hives, for medium colonies, the oxalic acid dose is 2g. The measuring spoon supplied, filled, but not pressed, corresponds to 1 gram of acid. If after the treatment you eliminate more than 1,000 mites, or in uncertainty about possible presence of the brood in the colony, it is necessary to repeat the treatment two weeks after, following the dose of 1g of acid, regardless of the size of the colony. Swarms and artificial swarms can be treated throughout the whole year. The nuclei with the queen must be treated before covering the brood. Warning: orphan colonies react with a frenetic "coming and going" with bees. For more details see the instructions in the information sheet on oxalic acid suitable for the use of beekeeping. The device is to be used exclusively in hives occupied by bees, do not treat weak colonies or empty hives. Below are the instructions for proper use. These indications must be applied by the user.

**The distance between the bottom of the hive and the frames should be at least 2cm.**

The vaporizer can be used in two different ways depending on the model, operator's experience, manual abilities and the security devices used.

**First mode - treatment time: 3 minutes**

(This mode is suitable for both the OXALIKA BASIC/START model and the OXALIKA PREMIUM model)

- wear personal protective equipment in accordance with Chapter I.
- ensure that other people within the area of the use of the device action are not within reach of the oxalic acid.
- ensure that the hive does not have any honeycombs.
- For a cold device, insert the correct dose of oxalic acid in the hollow of the cup with the measuring cup, taking into account the instructions on the drug label.
- Place the cap deeply through the opening for the bees. Do not place the cup too close to the wax but at a minimum distance of 2 cm.
- Remove any remnants of wax from the bottom of the

hive.

- If you cannot put the cap in the opening because of build of the hive, you can still use the drawer opening.
- Seal the opening with a damp cloth or wet foam
- All open areas should be sealed, over the entire surface
- Connect the terminals to the battery. Keep the source of energy connected to one of the two clamps. Through the contact of the second clamps, it can be switched on and off. For the operation of the burner, the red and black clamps can be applied without distinction to the positive or negative pole.
- Wait for 3-4 minutes in a sheltered position from any acid vapors that may come from the hive.
- Disconnect the terminals from the battery
- After the burner is off, wait 2 minutes before removing the appliance from the hive, so it can facilitate the deposition of the oxalic acid vapors.
- Extract the burner and tightly reclose the opening and immerse the cup (and only that) in water. This way you avoid the cooktop starting to evaporate the acid before being inserted in the hive when filling a new cup.
- To prevent lime scale buildup, wipe the cup with a dry cloth before the next treatment.
- After 10 minutes, remove the fabric from the opening.
- Repeat the operations from the beginning for further treatment of other hives.
- In case of any malfunctions and / or damaged parts, immediately stop using it and send it to the manufacturer for a maintenance intervention.

**Second mode - treatment time: 1,5 minutes** (for experienced users)

**Warning! Follow this mode only if you have the OXALIKA vaporizer with the temperature control "Premium or 130 / E"**

Unlike the first mode, in this mode:

- The operator must wear an ABE / P3 filter mask as specified in chap. I besides leather gloves and well adhering goggles.
- The sublimator must always be connected to the battery and should not be cooled between a treatment and the next.
- The operator will be located near the opening of the hive, inserting the acid into the bowl and immediately inserting the bowl inside the hive.
- Wait for 1 minute and 30 seconds (and in any case until the end of sublimation) before extracting the sublimator from the hive

The other indications given in the description of the first mode of operation are valid.

#### IV. Storage

After use place the device in a dry place and away from direct sun light.

#### V. Maintenance

##### Sequence of operations for maintenance

- Switch off power
- Check the general condition of the machine
- Clean with water, making sure to only wet the metallic block with recess to prevent water from entering the internment handle.

##### Replacing the fuse inside the handle.

N.B. : This should only be performed by a trained personnel in the field of electrical installations.

- Switch off power
- Loosen the end of the handle nut
- Take the rear and close the fuse holder.
- Open the fuse and replace the fuse with an identical one
- Close the device.
- In the case that, despite the replacement of the fuse, the unit does not resume functioning correctly, contact the manufacturer. Do not make any modifications to the product.

#### VI. Packaging and Disposal

The packaging is made of recyclable material. At the end of its use, lay it in a special container. The burner, as all electrical and electronic equipment, should not be disposed of with the household waste. Contact your local disposal service to have information about the waste collection system in your area.

#### VII. Product liability limits

Before use, carefully and scrupulously, read the present user manual completely. We disclaim any kind of liability in case of insufficient efficacy or failure, due to non-compliance with the user requirements and circumstances on which TALITHA SRLS has no influence.

#### VIII. Declaration of conformity

Manufacturer: TALITHA SRLS - Via Ugo Foscolo 21 / A, 57025 Piombino (LI) declares that: The oxalic acid vaporizer MODEL: OXALIKA:

complies with the directives:

- 2011/65 / EU (RoHS)
  - 2014/30/EU (Electromagnetic Compatibility Directive)
- PIOMBINO, December 20, 2016

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